

IVD

A photograph showing a man and a young girl from the chest up. Both are wearing face masks. The girl is on the left, wearing a yellow jacket and a yellow face mask. The man is on the right, wearing a dark blue shirt and a light blue face mask. They are looking at each other against a bright, hazy background, possibly outdoors during sunrise or sunset.

Nasale
Probenentnahme

Ninonasal[®] 5 professional

- Antigentest für schnelle SARS-CoV-2 Diagnostik

Patientenfreundlich, schnell und verlässlich

- Schmerzfreie Probenentnahme: besonders gut für regelmäßige Anwendung
- Schnelle Ergebnisse: nach 15 Minuten wissen Sie, ob die Person Corona positiv ist
- Exzellente Performance: Sensitivität 98% und Spezifität 99%

Der Ninonasal[®] sagt zuverlässig aus, ob jemand mit Corona infiziert ist, oder nicht. Dabei weist er auch die britische, südafrikanische und brasilianische Variante nach.

COVID-Diagnose auf einen Blick

In einer Pandemie ist es wichtig, möglichst viele Menschen regelmäßig zu testen, um die Ausbreitung des Virus zu reduzieren. Gerade große Kohorten benötigen einen Test, der einfach in der Handhabung ist und zuverlässige Ergebnisse liefert.

Der Ninonasal® 5 professional für den qualitativen Antigennachweis ist wie dafür gemacht. Die Probenentnahme und das Auftragen der Proben ist unkompliziert und dauert maximal 5 Minuten. Nach nur weiteren 15 Minuten steht fest, ob der Virus nachgewiesen wurde, oder nicht.



Ninonasal® 5 professional Performance mit der RT-PCR verglichen:

	RT-PCR		
	Positiv		Negativ
Ct* value	Ct <30	Ct <34	-
Positiv	113	125	2
Negativ	0	3	388
Total	113	128	390
	Sensitivität		Spezifität
	100%	98%	99%
	IC 95% [97-100%]	IC 95% [93-100%]	IC 95% [98-100%]

* Cycle threshold

Entwickelt, registriert und hergestellt in Europa - vollständig CE IVD konform

Für mehr Informationen wenden Sie sich bitte an folgende Adresse:

Hain Lifescience GmbH - a Bruker company
Phone: +49 (0) 7473 9451 0
info.mdx.de@bruker.com

In countries, where the IVD-CE mark is not applicable, the Ninonasal® 5 professional is subject to local regulatory approval. Ninonasal® 5 professional is a registered trademark of NG Biotech®.

Please contact your local representative for availability in your country.
Not for sale in the USA.



As of May 2021, Bruker Daltonik GmbH is now Bruker Daltonics GmbH & Co. KG.



Guipry · France
Phone +33 (0) 2 23 30 17 83

Bruker Daltonics GmbH & Co. KG

Bremen · Germany
Phone +49 (0) 421-2205-0

info.md@bruker.com - www.bruker.com/microbiology